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## **REPLACEMENT PARTS FREE OF CHARGE (WARRANTY)**

We will provide replacements for all parts for 36 months, except for the fastening belt, comfort bands and spare parts with expired warranty (see our general terms and conditions).

- Check the date on the receipt to make sure your PeniMaster® is no older than 3 years.
- Send the defect part, with sufficient postage, together with a copy of the receipt to the address given below.  OR
- Make a clearly legible digital copy of your receipt (e.g. with a scanner) and a photograph of the defect part in which the defect is clearly recognizable. Send both documents as .jpg and/or .gif files to: ➔ [www.penis.de](http://www.penis.de)

## **FASTENING BELTS, COMFORT BANDS AND REPLACEMENT PARTS WITH EXPIRED WARRANTY**

Please use the following internet address to order replacement parts for devices without warranty, fastening belts or comfort bands: [www.penis.de/spare-parts](http://www.penis.de/spare-parts)

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## CONGRATULATIONS!

Dear customer!

Thank you for purchasing PeniMaster®. You have selected the only recognised principle for permanently enlarging the penis painlessly and without risky operation. If you follow these instructions on how to use PeniMaster®, your penis will become longer and thicker both when flaccid and erect. PeniMaster® is also suitable for correcting unwanted curvature of the penis and may even be used to extend the foreskin (e.g. for circumcised users).

Improved blood flow and training the muscles of the penis in many cases leads to increased pleasure, longer-lasting erections and greater sexual stamina. The success they have had with PeniMaster® has helped many users overcome the complexities that have prevented them from enjoying a free and natural sex life for many years. Finding a partner also often comes easier knowing that you physically "stand out".

You can easily use PeniMaster® alone without needing help from a third party (e.g. a physician). PeniMaster® produces the same good results whether the user is circumcised or not. PeniMaster® is a professional device and is suitable for all ages. However, we only recommend its use for adults in order to ensure that the penis has reached its natural length before beginning treatment.

## HOW IT WORKS

The principle of PeniMaster® is simple: Continuous traction applied over an extended period of time stimulates the cell tissue in the penis to grow: New cells are created, making the penis longer and thicker. Irregular distribution of tissue in the penis – the cause of bending in the penis – is balanced out with new tissue: The penis becomes straighter. With PeniMaster® the foreskin can be stretched more than the rest of the penis. This means that an elongation of the foreskin is also possible if desired (see instructions).

*Note: Although the penis appears thinner during stretching, it will return again to its original thickness or will be thicker after use!*

Modern medicine also uses this principle of growth by traction in orthopaedics and skin surgery, for example, where special traction equipment is also employed.

The idea of lengthening the penis by permanent traction is certainly nothing new. The Karamojong in northern Uganda and the Sadhus in India achieve impressive penis lengths by attaching weights to their penises (see Encyclopaedia Britannica; Stern No. 49/95).

## USING PENIMASTER®

### APPLICATION

The success of PeniMaster® depends on both the traction exerted on the penis and the duration of use. Only a good balance between traction and duration of use will guarantee the best results.

[!] *Generally speaking, wearing PeniMaster® should always be a pleasant experience and should not cause any pain at all! Pain is a definite indication of incorrect use, caused, for example, by setting the strength of traction too high or fastening too tightly!*

### PREPARATION FOR PUTTING ON PENIMASTER®

- Pull the supporting surface away from the supporting rods.
  - Pull the fastening belt out of the slits in the supporting surface.
 

**IMPORTANT: Push both locking wings down onto the underside of the bearing surface. If this is not done, the fastening belt may be damaged. This also applies to removal after use or if the hold on the penis needs to be loosened using the fastening belt.**
  - Screw the fine adjusting screws to the lowest level (until the thread can no longer be seen).
  - Select the correct total length of the device (corresponds approximately to the length of your erect penis plus 1.5 cm). Remove, connect or replace the extension rods supplied to achieve the basic length of the device to fit you. The extension rods are 2, 4, 6 and 8 cm long and can be connected to each other. This provides the following extension lengths: 10 cm (extension rods 8+2); 12 cm (8+4); 14 cm (8+6); 16 cm (8+2+6); 18 cm (8+4+6); 20 cm (8+2+4+6).
- [!] *The six mini-rods (supplied as sets of two rods, each 1.5 cm long and screwed together) are used for fine adjustments of +0.5, +1.0 or +1.5 cm.*

### PUTTING ON PENIMASTER®

- Elongate the penis by hand for about one minute. Guide the penis while flaccid through the attachment.
- FASTENING METHOD I** – For more pronounced elongation of the foreskin: To elongate the foreskin more noticeably or more in relation to the penis, (e.g. to regain foreskin on a circumcised penis), place the comfort band on the penis over the foreskin, if necessary at a sufficient distance from the gland. This will stretch the skin more intensely and thus stimulate its growth more strongly.
 


**FASTENING METHOD II** – For less pronounced elongation of the foreskin in proportion to the penis: To make the foreskin grow less pronouncedly or in proportion to the penis, pull back the foreskin and place the comfort band on the penis directly behind the gland. This will stretch the skin less and will provide less stimulation of growth than in FASTENING METHOD I.
- Place the fastening belt over the comfort band so that the anatomically formed curvature of the fastening belt (between the two "ears") points towards the gland. *The comfort band can be wrapped around the penis as many times as is comfortable, otherwise it can be shortened to an appropriate length or can even be left out completely.*
- Push both ends of the fastening belt back through the slits in the supporting surface. Put a finger between the penis and the supporting surface to avoid pinching skin in the slits. Tighten the ends of the fastening belt firmly and then remove your finger.
- Your penis is now fastened to the supporting surface
 

[!] *Avoid excessive pressure on the penis!* Should the penis or gland change colour, immediately loosen the fastening belt and massage the penis to re-activate the blood circulation. During the first few days of use, take particular note of any negative changes in the appearance or functioning of the penis and discontinue use until the symptoms have abated.
- Push the supporting surface that holds the penis to the end of the rods.
- You have the option of wearing the device either upwards or downwards, achieving the same good results either way.
- Adjust the traction of PeniMaster® by loosening the screw thread (see also next point "adjusting the traction").

## ADJUSTING THE TRACTION

There are **5 NOTCHES ON THE END OF EACH METAL ROD.**

Following the principle of spring balance, they indicate the strength of traction applied to the penis.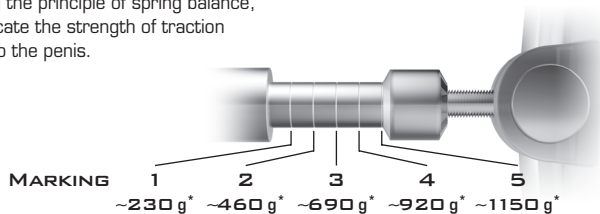

This is an example of how to read the spring balance: Once you are wearing the device, if it extends until only 3 of the 5 markings are still visible, the traction is set between 460 and 690 grams. If only the last marking is still visible, the traction is between 920 and 1150 grams. Only use a weak traction to begin with to get used to the device (e.g. between 0 and 230 g).

To increase the traction on the penis, twist both adjustment screws evenly in an anti-clockwise direction. This will increase the overall length of the device. To decrease the traction of the penis, turn the adjustment screws in the other direction (clockwise).

*Note: The screw thread can only be unscrewed when the penis is slightly stretched by selecting the correct rod length (see above). Otherwise no traction, or an insufficient amount of traction, will be created! Select a low level of traction when beginning treatment (e.g. between 0 and 230 g) in order to get used to the device. This can take up to two weeks. You will best train your penis if you always select the highest possible traction with which you can wear the device for 60–90 minutes without any problems.*

*Note: Should the penis stretch further some time after application, therefore resulting in weaker traction, please readjust the traction accordingly! Increase the traction slowly over the course of a few weeks until only one marking remains visible on the spring balance if possible. If you start to feel any pain, please set PeniMaster® at a lower traction!*

### EXCESSIVE OR PAINFULL FASTENING OR TRACTION CAN HAVE A NEGATIVE EFFECT ON CELL GROWTH.

[!] For your safety, you should not set the device to loads over 1150 grams, even if you feel this would not cause you any discomfort.

## HOW LONG TO WEAR PENIMASTER®

PeniMaster® should be worn for at least three hours daily. Wearing it for longer periods improves the results! Every 60 to 90 minutes, take a 15 minute break and massage the penis to activate the blood circulation. These short breaks support the growing process!

After a sufficient period of getting used to PeniMaster®, you can wear it under loose clothing (please remember to take breaks every 60 to 90 minutes!).

Wearing it for 6 or more hours per day leads to optimal growth results.

In principle, there is no maximum number of months you can use PeniMaster®. On the other hand, the longer you wear PeniMaster®, the more noticeable the result. The minimum wearing time for visible success is around 3 hours a day for a period of one month. Results which are achieved in the first four weeks can be due to tissue expansion and may not be permanent.

Since no break can be taken from wearing the PeniMaster® during sleep, use whilst sleeping is not recommended and should only be considered if a sufficiently effective period of time cannot be achieved otherwise. Before using it while sleeping, the user must have a sufficient amount of experience with the settings that are right for him for use over several hours whilst awake.

Do not worry about missing out on good results if you sometimes cannot wear PeniMaster® for a few days or even over a longer period, for example while on holiday. Growth that has already been achieved is retained and continues once you resume treatment. Please remember that the method helps your penis to grow – and, naturally, this takes time! Take an optimistic view and do not lose patience if you do not see any results after just a few days. A larger penis will be your reward for your perseverance!

## HYGIENE ADVICE

In order to enjoy your PeniMaster® for a long time, Please clean penis and PeniMaster® before and after each use with lukewarm water and soap. Dry the penis and device thoroughly. This serves general hygiene, prevents infection and also makes it easier to fasten the penis with the fastening belt. This is more likely to slip off if there is any body grease, powder or oil-based care product present.

The device is rust-free and is not damaged by contact with water.

## PRECAUTIONS FOR USE AND WARNINGS

Use of PeniMaster® is not known to produce any remaining side-effects. Slight reddening of the skin in the area of the supporting belt is normal during use due to mechanical load. This is not painful and the reddening quickly disappears when PeniMaster® is taken off.

After using PeniMaster®, it is recommended to rub moisturising lotion onto the penis to relax the skin.

Please observe the following precautions and warnings:

### PENIMASTER® SHOULD NOT BE USED

- under the influence of alcohol
- under the influence of narcotics
- under the influence of sedatives
- under the influence of sleeping drugs or drugs that lower awareness of pain
- in the case of a known chrome allergy or intolerance of polycarbonate (PC) and TPE
- in the case of known problems with the penis, the testicles or the urethra
- in the case of weakness of the veins or connective tissue
- in the case of existing pain or the onset of pain in the penis or testicles
- together with penis implants

Should pain, changes in colour or other kinds of external irritation or functional changes of the penis, gland or testicles suddenly occur while using PeniMaster®, stop the treatment immediately and do not continue until symptoms have abated. Should the symptoms continue or worsen after discontinuation of treatment, consult a physician or urologist as a matter of precaution. If, owing to physical limitations or a known disease, you are not sure whether use of PeniMaster® is suitable for you, please consult your physician or urologist before using it.